

Quick Start Guide

for Using the
Logix Smart™ SARS-CoV-2 Direct Saliva IVD Product
 (COVDS-K-003 and COVDS-K-004)



1 Plate the Master Mix

Perform in Clean Area

- a. Load the Co-Dx Box software.




- b. In the Clean area, clean work surfaces with water > 10% bleach > water > 70% alcohol (in that order), or another equivalent method of cleaning that disinfects and degrades nucleic acids.



- c. Thaw the master mix (MM) and the nuclease-free water (NFW) (for the no-template control [NC]), on ice.




- d. Briefly vortex and centrifuge the MM and NFW reagents.

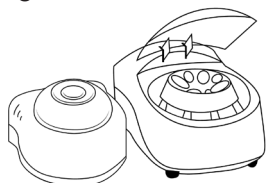


Perform in Clean Area


- e. Add 10 μ L of MM into every well that will be used (1 well for NC, 1 well for positive control [PC], and 1 well for each sample).

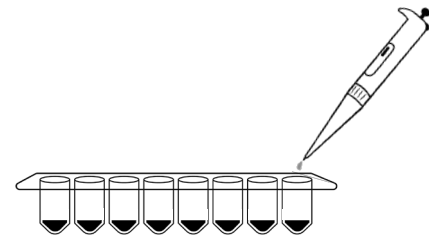

- f. Pipet 10 μ L of NFW into only the NC well and cap the NC well.

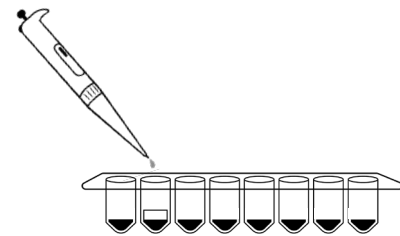

- g. Place reagents back into the freezer.

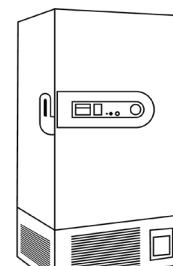

Quick Start Guide

(continued)

for Using the
 Logix Smart™ SARS-CoV-2 Direct Saliva IVD Product
 (COVDS-K-003 and COVDS-K-004)



2 Plate Samples and Positive Control (PC)

Perform in Positive Area

a. Load the Co-Dx Box software.




b. In the Positive area, clean work surfaces with water > 10% bleach > water > and 70% alcohol (in that order).



c. Thaw the PC on ice.




d. Briefly vortex and centrifuge all reagents.

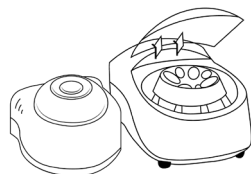

e. Perform sample extraction, either using RNA extraction or lysis.

Perform in Positive Area

f. Add 10 μ L of the sample extract to the appropriate wells and cap the sample extract wells.

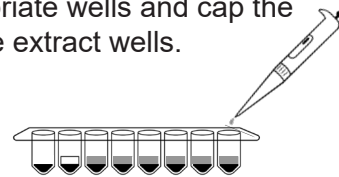


g. Add 10 μ L PC to the PC well and cap the PC well.



h. Place capped tubes in the thermocycler, in appropriate locations and fill the rest of the carousel with blank tubes.



i. (Optional): Cap all tubes and place samples and reagents back into the freezer.

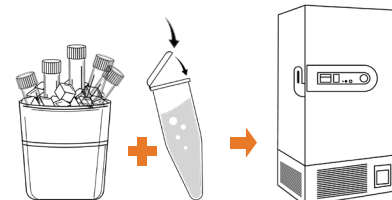

Quick Start Guide


(continued)

for Using the
 Logix Smart™ SARS-CoV-2 Direct Saliva IVD Product
 (COVDS-K-003 and COVDS-K-004)




3 Start the Run


a. In the Co-Dx Box software, click **New**.

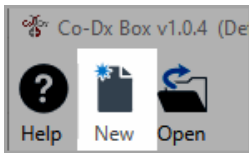


b. Select an assay template and click **OK**.

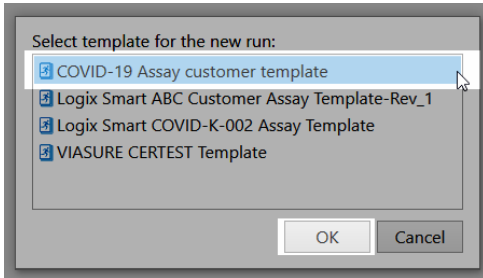


c. Under **Run Setup**, select **Samples** and type sample information in the appropriate order.

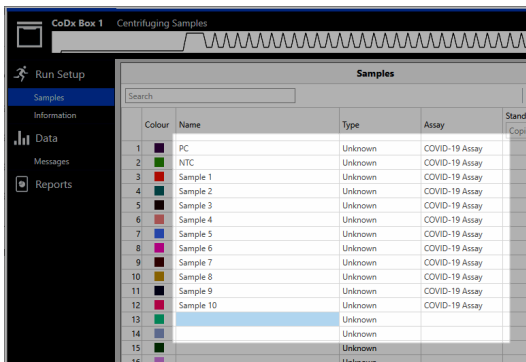


d. Click **Information** and in the **Notes** field, type any needed information.

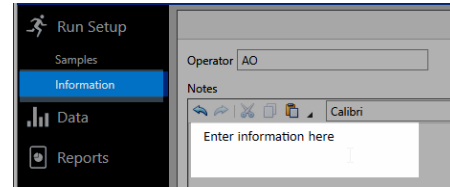

e. Select the appropriate thermocycler and from the drop-down select **Start Run**.

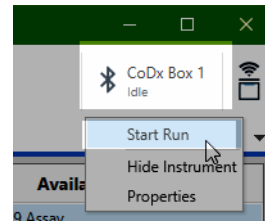

f. Verify the tube clamp is in place and the appropriate reaction volume is listed, and select **Start**.

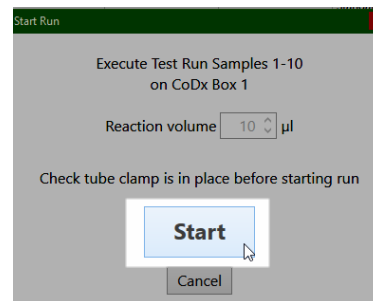

The run initializes and a progress bar gives an estimated time to completion.

Perform in Co-Dx Box Software

Perform in Co-Dx Box Software